**VENOSAN®**

Compression Stockings

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## INSTRUCTIONS FOR USE

Thank you for choosing VENOSAN® compression stockings. These have been produced in accordance with the latest medical knowledge. The stockings are a medical device, and it is therefore absolutely essential that you read these instructions very carefully.

### **Product description and features**

Compression causes the leg vein diameter to reduce and enables the vein valves to close again. The backflow velocity of the blood to the heart is thus increased and the blood no longer sinks into the legs.

### **Material composition**

You can find precise details on the material composition of your VENOSAN® compression stocking on the sew-in textile label and on the back of the packaging.

### **Intended purpose**

Long-term treatment of phlebological and/ or lymphatic diseases in the case of legs.

### **Indications**

**Varicosis:** Primary and secondary varicosis, varicosis during pregnancy, after sclerotherapy, after vein surgery

**Thromboembolism:** Thrombophlebitis (superficial) as well as after treatment of phlebitis,

deep venous thrombosis, after thrombosis, post-thrombotic syndrome, prophylaxis of thrombosis for ambulatory patients

**Chronic venous insufficiency:** CVI grades I to III according to Widmer, C1S-C6

according to CEAP, prevention of venous ulcers, insufficiency of the communicating veins, angiodyplasia

**Oedemas:** Lymphoedema, oedema during pregnancy, post-traumatic oedema, postoperative oedema,

idiopathic cyclic oedema, grade II lipoedema and above, congestive conditions following prolonged immobility (arthrogenic stasis syndrome, paralysis and partial paralysis of the limb).

**Other:** After burns, hypertrophic scarring

### **Contraindications**

Known allergy and/ or hypersensitivity to any of the product components. VENOSAN® medical compression stockings must not be worn by anyone suffering from the following:

**Absolute:** Advanced peripheral arterial occlusive disease, decompensated heart diseases, septic phlebitis, phlegmasia coerulea dolens

**Relative:** Concomitant dermatoses, intolerance to compression stocking fabric, sensory disturbances of the limb, advanced peripheral neuropathy, primary chronic arthritis.

Failure to observe these contraindications will result in the invalidation of any liability claims.

### **Risks and side effects**

The use of stockings with silicone self-supporting tops and silver may lead to skin irritation and/ or allergies may occur in rare cases of sensitive skin. Medical compression stockings can cause skin necrosis and pressure damage on the peripheral nervous system, especially if misused.

### **Warnings and Precautions**

Be sure to consult trained medical personnel before wearing your stockings for the first time. The stockings should not be worn on open wounds. It is very important that the stockings are the correct size and fit you exactly. Never fold over the top seam. This will double the pressure which can lead to dangerous constriction. If the pain in your legs increases when you are wearing your stockings, or if you feel unwell, you must consult your doctor or a suitable specialist immediately. Please make sure you pay attention to well-groomed feet and a good footwear, so that the fine knit of the stockings will not be damaged. Certain creams, ointments and oils can destroy your stockings or leave unsightly stains. For hygienic reasons, this product is not suitable for treating more than one patient. The textile label is an important part of the compression stockings. If you remove this label, the claim for warranty or exchange is void. If you use and care for your stockings according to the instructions they will have a useful life of six months. After this period the medical efficacy can no longer be guaranteed and we recommend you buy new stockings.

## **Application instructions**

### **1. Preparation**

Wash before use! Sharp or cracked finger nails, rough skin, sharp-edged rings, bracelets, or wristwatches can damage the sheer knitted fabric of your VENOSAN® when you are putting them on or taking them off.

### **2. Application**

The initial fitting of the product must only be performed by medically trained specialists.

Please note, compression stockings should be carefully applied and not simply pulled up the leg. It is best to put on your stockings first thing in the morning, as soon as you get up. Your feet and legs must be dry. Talcum powder can be helpful if necessary.

1. Turn the leg of the stocking inside out down as far as the heel, i.e. except for the foot, the inside of your stocking must be facing outwards.
2. Use both thumbs to stretch the foot of the stocking open and pull your stocking over your foot.
3. Grip the upper edge of the stocking and pull it over your heel.
4. Once all the leg of the stocking is above your ankle, reach inside with both thumbs and massage the fabric up your leg with a zigzag movement.

Since your VENOSAN® stockings are made of highly elastic material it is very important that you do not pull the top seam up as hard as you can. The stocking has to be massaged upwards in a series of side-to-side movements.

1. When removing the stockings, pull the top seam of the leg part downwards as far as your ankle.
2. Push both thumbs between the knitted fabric and your leg and ease it over your heel.
3. Pull the stocking slowly over your foot.

Wear your compression stockings every day. Therapeutic success can only be ensured by wearing the stockings regularly.

Please consult your doctor or chemist immediately if pains or severe skin irritations occur while wearing the compression stocking.

### **3. Care recommendation (care instructions)**

You can wash your VENOSAN® compression stockings in the washing machine at max. 40°C on a delicates programme. Do not use any fabric softeners, optical brighteners or stain removers (such as turpentine, benzine etc.) To maximize the product lifespan do not use a tumble dryer or radiator for drying (see label). Clean the silicone bands on self-supporting tops with alcohol regularly to improve adhesion.

### **Storage and transport conditions**

Store the stockings at room temperature and protect against humidity and direct sunlight. You can find the expiry date of your compression stocking in the form of an hourglass on the packaging label.

Please note that the service life of compression stockings is 60 months when stored properly.

Our custom-made products are intended to be used immediately, so no maximum storage date is indicated on the packaging.

### **Disposal**

The VENOSAN® compression stockings can be disposed of in household waste. If deviating national statutory provisions exist, these must be applied.

### **General information**

In case of serious incident, contact the manufacturer and the responsible health authorities.

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SWISSLASTIC AG ST. GALLEN