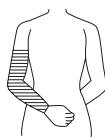
VENOSAN®

Compression Stockings

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Thank you for choosing VENOSAN® compression armsleeves. The sleeves were produced based on the most recent advances in medicine. The product is a medical garment. It is therefore essential that you read these instructions for use thoroughly.

Product description and performance features

The compressive knitted fabric increases pressure on the tissue layer of the arms, i.e. the upper arm, improving the transport capacity of the regional lymphatic vessels. This prevents renewed swelling (recurring oedema) of the extremities and stabilises the treatment success. The breathable, skin-friendly knitted fabric ensures a physiological lymphatic flow by means of decreasing compression from the hand to the shoulder joint. We recommend that you wear the arm sleeve daily in order to preserve optimised decongestion of the oedema.

Material composition

Refer to the sew-in label and the rear of the packaging for the exact specifications of the material composition.

Intended purpose

Your compression arm sleeve helps to prevent oedema occurring.

Indications

Post-operative and post-traumatic oedemas (please note: not suitable for severe oedemas!), shoulder arm vein thrombosis

Contraindications

Known allergies and/or hypersensitivity to any of the product components.

The medical compression arm sleeves must not be worn by patients experiencing any of the following conditions. **Absolute:** Peripheral arterial occlusive disease (PAOD) stage III/IV, congestive heart failure, septic phlebitis, acute deep thrombosis of the extremities, phlegmasia coerulea dolens, erosive dermatosis, severe erysipelas, acute bacterial, viral or allergic infections, untreated or advanced contact eczema, untreated open wounds, swellings of the extremities with unknown causes, severe oedema.

Relative: Peripheral arterial occlusive disease (PAOD) stage I/II, skin diseases (e.g. cystic, forming or developed weeping dermatosis), malignant lymph oedema. Advanced peripheral neuropathy (e.g. diabetes mellitus), advanced sensory disorders of the extremities, compensated heart failure, primary chronic polyarthritis, allergic reactions to the compression sleeve material. Any and all liability will be rejected if these instructions are not followed.

Risks and side effects

Persons wearing arm sleeves with silicone and silver adhesive tapes may experience skin irritation if they are hypersensitive to these materials.

Warning notice and precautionary measures

Consult with a specialist before you wear the arm sleeves for the first time. The arm sleeves must not be worn on open wounds. It is important that the size and fitting are exactly correct. Never fold down the top edge of the sleeve. Doing so would double the pressure, which might lead to dangerous congestion. Consult with your doctor or a specialist immediately if the pain in your arm increases or if you feel unwell when wearing the sleeve. Some ointments and oils can destroy your sleeves or leave ugly stains. For hygienic reasons only one person should wear this product. The fabric label is an important part of the compression arm sleeve. You will void all warranty claims and claims to replacement if you remove this tag. The sleeves can be worn for six months, provided they are used and cleaned according to these instructions. Their medical effectiveness cannot be guaranteed for longer periods, and we recommend purchasing a new compression arm sleeve after the six months of wear have elapsed.

Wearing instructions

1. Preparation

Wash before use! Sharp or cracked fingernails, rough hands, rings with sharp edges, bracelets, watches and suchlike may damage the delicate knitted fabric of your compression arm sleeves when putting them on or taking them off. Use suitable dressings or a bandage to cover any open wounds before putting on the sleeve.

2. Application

The first fitting of the product must only be carried out by a medical professional.

How to apply a compression arm sleeves with knitted cuff or adhesive edges:

1. Fold the top half of the sleeve over the bottom half of the sleeve.
2. Pull the folded sleeve over the hand and up the arm, until the edge of the fold is level with the elbow.
3. Ease the folded section of the sleeve over the elbow and then gradually roll it up until it reaches the shoulder.
4. Distribute the knitted fabric of the sleeve to eliminate any creases and make sure it sits properly along the whole length of the arm. If the sleeve has been applied correctly, there should be no creases and it will not restrict movement.

How to apply a compression arm sleeve with shoulder cap and strap:

1. Place the shoulder strap across the chest and the other strap over the back. Then connect the ends of the two straps. Attention: Use the adjustment knob to adjust the length if the shoulder straps are too loose or tight.

Consult your doctor or pharmacist immediately if you experience pain or skin irritation when wearing the compression arm sleeve.

3. How to care for your compression arm sleeve

Machine wash your VENOSAN® compression arm sleeve daily at a maximum of 40° C on a delicate wash cycle. For washing, use a mild detergent without fabric conditioner, colour brightener or stain remover (such as turpentine, white spirit etc.) You are advised to refrain from placing the sleeve in a dryer or on a radiator to ensure a maximum product lifespan (follow the instructions on the care label)

Occasionally wipe the inside of the silicone adhesive tape with alcohol to improve adhesion.

Storage and transport conditions

Store the sleeves at room temperature and away from moisture and direct sunlight. The shelf life when properly stored is indicated by an 'hourglass' on the packaging label. The shelf life is 60 months.

Custom made medical compression arm sleeves are products for immediate use and therefore do not have a shelf life date.

Disposal

You can dispose of your VENOSAN® compression arm sleeves in household waste. Please observe the local rules and regulations regarding disposal of waste products in your country.

General instructions:

Contact the manufacturer and the relevant health authorities if a serious incident occurs.

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TACTEL® fibre is a registered trademark
SWISSLASTIC AG ST. GALLEN